# Norco® Universal Cuff

**Intended Purposes** Non-sterile device used to hold objects during daily activities such as eating utensils, a toothbrush, pen or small handled hair comb.

**Indications** Persons with limited finger range of motion, dexterity or weak grasp.

**Contra Indications** Skin irritation, allergy to polyester or spandex.

## Instructions for Use / Proper Fit

Unfasten or loosen the strap. Slide handle of object completely into the pocket. Loosen d-ring strap and slip hand in, so the pocket fits comfortably in the palm. Secure the d-ring strap by wrapping the hook section across the top of the hand and attach to loop. Fit should be comfortable. If tight, loosen strap.

Note: D-ring on end of strap may aid to secure or loosen the strap for those users with limited finger strength or dexterity.

**Warning:** If cuff feels too tight, skin becomes irritated, or if pain, stiffness, or numbness occur, discontinue use and seek advice from a healthcare professional.

**Instructions for Care** Hand wash in cool water and mild soap. Air dry. Do not place in dishwasher.

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North Coast Medical, Inc.
Morgan Hill, CA 95037 – U.S.A.
ICON (LR) Limited
Allenspach Medical AG